



Accutest® Phenaphthazine Paper QC Procedure

Each lot has been verified for accuracy using NIST traceable standards. Expired rolls should be disposed of. Unopened, unexpired rolls do not require further QC testing, nor do rolls that have been opened less than 30 days. Rolls that have been opened more than 30 days may be retested to ensure proper performance by following these procedures using NIST traceable buffers.

Quality Control Procedure

1. Tear off a strip of paper from the roll.
2. Dispense a drop of the pH 5.0 buffer solution onto the strip of pH paper.
3. Immediately match the color of the pH paper with the closest color on the color chart supplied with the paper dispenser.
4. Repeat steps 1-3 with the pH 7.0 buffer solution, or with the buffer required by your testing protocol.

Note:

Color comparison recommended under a combination of fluorescent light and daylight. The color of the base stock may vary from lot to lot, from tan to olive green. This will not affect the accuracy of the results.

JANT PHARMACAL CORPORATION
LABORATORY & POINT OF CARE DIAGNOSTICS

JANT PHARMACAL CORPORATION
16530 VENTURA BLVD, SUITE 512 ENCINO, CA 91436
PHONE: 800.676.5565 818.986.8530 FAX: 818.986.0235
info@jantdx.com www.jantdx.com



Product Description:

Accutest Phenaphthazine paper is an indicator paper intended for in-vitro, semi quantitative determination of pH in the 4.5 – 7.5 range.

Distributed by:
Jant Pharmacial Corp:
Encino, CA 91436
Toll Free: 800-676-5565
www.jantdx.com

Results:

Visual comparison of colors between the test paper and the color chart will yield results in clinically meaningful units. Any resultant pH reading should be considered along with other laboratory and clinical findings to derive a final diagnosis.

Storage and Shelf Life:

Product should be stored at room temperature, out of direct sunlight. Unopened Accutest pH Phenaphthazine Paper should be used by the printed expiration date. Once opened, shelf life is approximately 6 months.

Precautions:

Protect against exposure to acid or alkaline fumes. Avoid excess heat and prolonged exposure to sunlight.

Color comparison recommended under a combination of fluorescent light and daylight.

Procedure:

Collect sample in a clean receptacle. Tear off a 2" strip from the roll. Dip into sample and immediately compare to color chart. Closest color match indicates resultant pH.

Quality Control:

For greatest accuracy, color comparisons should be conducted in a combination of fluorescent light and daylight. Each lot of color charts and Accutest pH Phenaphthazine Paper is calibrated using NIST traceable buffer standards under the light conditions noted above. Note that base stock color may vary from lot to lot, from tan to olive green. This will not affect the accuracy of the pH reading.