

## Status<sup>®</sup> Flu A & B Control Set

For *In vitro* diagnostic use

Cat. No.: 36022CS

### Intended Use

Status<sup>®</sup> Flu A & Control is to be used as an external control to ensure the performance of the following test kits. These controls are used to ensure proper performance of these tests:

- Status<sup>®</sup> Flu A & B (Cat. No.: 36025/36022/36005)
- BioSign<sup>®</sup> Flu A + B (Cat. No.: BSP-510-25/BSP-510-22)
- McKesson Influenza (Cat. No.: 181-36025)

### Summary and Principle

The Status<sup>®</sup> Flu A & Positive Control Swab consists of recombinant nucleoproteins of influenza A and B viruses dried onto the swab. The Negative Control Swab consists of inactivated Group B *Streptococcus* dried onto the swab.

### Materials Supplied

- Flu A & B Positive Control Swabs (5): Influenza A and B antigens (non-infective recombinant nucleoprotein)
- Flu A & B Negative Control Swabs (5): Inactivated Group B *Streptococcus* antigen (non-infective)
- Package Insert (1)

### Precautions/Warnings

- For *In vitro* diagnostic use.
- Do not use after the expiration date.
- Controls contain recombinant protein and inactivated microorganisms. However, handle as if capable of transmitting infectious disease.
- Wear disposable gloves while handling these controls.
- Dispose of used controls in accordance with Federal, State, and Local requirements.
- Follow proper hand washing hygiene after handling these controls.
- The Control Swabs are designed for use only with the tests listed above.

### Stability and Storage

Status<sup>®</sup> Flu A & Control Set may be stored at 2-30°C (35-86°F) in the original sealed pouch, away from direct sunlight. The contents are stable until the expiration date printed on the pouch or box.

### Test Procedure

Refer to the package insert for the test.

The Status<sup>®</sup> Flu A & B Control Swabs must be used at room temperature (18-30°C)

### Interpretation of Results

Refer to the package insert for the test.

### Technical Assistance

For technical assistance and customer service, contact:

LifeSign, LLC  
85 Orchard Road  
Skillman, NJ 08558  
Phone: (800) 526-2125 Fax (732) 246-0570  
www.lifesignmed.com