



Driven to
perform.



OraQuick ADVANCE® Rapid HIV-1/2 Antibody Test

The early HIV antibody detection
test for recent infection*

FDA-APPROVED DATA comparable
to 3rd-generation lab-accurate results

OraQuick Rapid
Antibody
Test
ADVANCE® **HIV-1/2**

Sensitivity. Versatility. Reliability.

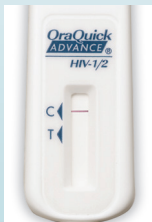
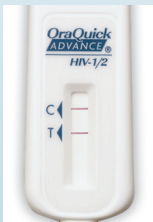
Sensitivity. Results with >99% sensitivity and specificity.*

Sample Type	Sensitivity	Specificity
Oral Fluid	99.3% (98.4 – 99.7%)	99.8% (99.6 – 99.9%)
Fingerstick	99.6% (98.5 – 99.9%)	100.0% (99.7 – 100.0%)
Plasma	99.6% (98.9 – 99.8%)	99.9% (99.6 – 99.9%)

Versatility. Testing platform suitable for both clinical and non-clinical settings.

CLIA-waived for oral fluid, fingerstick, and venous whole blood; moderately complex for plasma.

Reliability. Rapid antibody testing results in 20 minutes.

Collect	Insert	Read
Step 1 Fingerstick whole blood sample. Wipe away first drop of blood. Use collection loop. 	Step 2 Insert device into vial containing buffer solution. 	Step 3 Read results between 20 and 40 minutes.   Non-reactive Preliminary Positive
Step 1 Oral Fluid sample. Swab the lower and upper gum once. 		

Driven to perform. **OraQuick** Rapid Antibody Test **ADVANCE® HIV-1/2**

Order Information

Description	CPT Code
Box of 25 tests	86703
Box of 100 tests	Medicare HCPCS
Controls	G0433
Visual Reference Panel	

*Refer to Package Insert for Information.

OraQuick ADVANCE® is a registered trademark of OraSure Technologies, Inc. ©2017 OraSure Technologies, Inc. Item#OQA0329 rev. 03/17



220 East First Street
Bethlehem, PA 18015 USA
phone: 1-800-ORASURE
www.knowyourstatustoday.com
web: www.OraSure.com
Made in the USA